
0050375 Measles (Rubeola) Antibodies, IgG and IgM

MEASLE PAN

Specimen Required: Collect: Serum Separator Tube (SST).

Specimen Preparation: Separate from cells ASAP or within 2 hours of collection. Transfer 1 mL serum to an ARUP Standard Transport Tube. (Min: 0.5 mL) Parallel testing is preferred and convalescent specimens **must** be received within 30 days from receipt of the acute specimens. **Mark specimens plainly as "acute" or "convalescent."**

Storage/Transport Temperature: Refrigerated.

Unacceptable Conditions: Refer to individual components.

Stability (collection to initiation of testing): After separation from cells: Ambient: 48 hours; Refrigerated: 2 weeks; Frozen: 1 year (avoid repeated freeze/thaw cycles)