

0099165

Glucagon

GLUCA



Specimen Collection and Handling

Specimen Required: Collect: Protease Inhibitor tube (PPACK; Phe-Pro-Arg-cholormethylketone) (ARUP supply #49662), available online through eSupply using ARUP Connect™ or contact ARUP Client Services at (800) 522-2787. A winged collection set must be used.
Specimen Preparation: Mix well. Separate from cells within 1 hour of collection. Transfer 1 mL plasma to an ARUP Standard Transport Tube. (Min: 0.5 mL)
Storage/Transport Temperature: Frozen. Separate specimens must be submitted when multiple tests are ordered.
Unacceptable Conditions: Grossly hemolyzed specimens.
Stability (collection to initiation of testing): After separation from cells: Ambient: Unacceptable; Refrigerated: 48 hours; Frozen: 3 months