
New Test **2013715** **LipoFit by NMR, Particle Count Only** **NMRLIPFITP**
Available Now

Methodology: Quantitative Nuclear Magnetic Resonance Spectroscopy
Performed: Sun-Sat
Reported: 3-6 days

Specimen Required: Patient Prep: 12 hour fast is preferred but not required.
Collect: Greiner Bio-One Clot Activator Tube (ARUP supply #53483). Available online through eSupply using ARUP Connect™ or by contacting ARUP Client Services at (800) 522-2787. Also acceptable: Plain red.
Specimen Preparation: Gently invert tube to mix contents and allow to clot at room temperature. Separate serum from cells within 8 hours. Transfer 2 mL serum to an ARUP Standard Transport Tube. (Min: 1 mL)
Storage/Transport Temperature: Refrigerated.
Unacceptable Conditions: Plasma. Serum separator tubes other than Greiner Bio-One.
Stability (collection to initiation of testing): Ambient: 2 days; Refrigerated: 1 month; Frozen: Unacceptable

Reference Interval: By Report

Interpretive Data: See Compliance Statement B: www.aruplab.com/CS

CPT Code(s): 83704

New York DOH approval pending. Call for status update.

HOTLINE NOTE: Refer to the Test Mix Addendum for interface build information.