

2013715

LipoFit by NMR, Particle Count Only

NMRLIPFITP

Specimen Required: Patient Prep: Fast 12 hours prior to collection.

Collect: Greiner Bio-One Clot Activator Tube (ARUP supply #53483). Available online through eSupply using ARUP Connect™ or by contacting ARUP Client Services at (800) 522-2787. Also acceptable: Plain red.

Specimen Preparation: Gently invert tube to mix contents and allow to clot at room temperature. Separate serum from cells within 8 hours. Transfer 2 mL serum to an ARUP Standard Transport Tube. (Min: 1 mL)

Storage/Transport Temperature: Refrigerated.

Unacceptable Conditions: Plasma. Serum separator tubes other than Greiner Bio-One. Non-fasting or lipemic specimens.

Stability (collection to initiation of testing): Ambient: 2 days; Refrigerated: 1 month; Frozen: Unacceptable