

---

2013716      **LipoFit by NMR**

**NMRLIPFIT**

**Specimen Required:** Patient Prep: **Fasting specimen required.**

Collect: Greiner Bio-One Clot Activator Tube (ARUP supply #53483) available online through eSupply using ARUP Connect™ or by contacting ARUP Client Services at (800) 522-2787. Also acceptable: Plain Red.

Specimen Preparation: Gently invert tube to mix contents; allow to clot at room temperature. Separate from cells within 8 hours.

Transfer 4 mL serum to an ARUP Standard Transport Tube. (Min: 2 mL)

Storage/Transport Temperature: Refrigerated.

Unacceptable Conditions: Plasma. Serum separator tubes other than Greiner Bio-One. **Non-fasting or lipemic specimens.**

Stability (collection to initiation of testing): Ambient: 24 hours; Refrigerated: 1 week; Frozen: Unacceptable