

2014704

Maternal T Cell Engraftment in SCID, Maternal Specimen

SCID-MAT



Time Sensitive



**New York Clients:** Direct Submission  
Instructions

**Specimen Required:** Collect: Lavender (EDTA), Pink (K<sub>2</sub>EDTA), or Yellow (ACD Solution A).

**New York State Clients:** Lavender (EDTA). **Collect Monday-Thursday only.**

Specimen Preparation: Transport 3 mL whole blood. (Min: 1 mL)

**New York State Clients:** Transport 14 mL whole blood. (Min: 6 mL). **Do not send to ARUP Laboratories.** Specimens must be received at performing laboratory within 24 hours of collection. For specimen requirements and direct submission instructions please contact ARUP Referral Testing at (800) 242-2787, ext. 5145.

Storage/Transport Temperature: Refrigerated.

Stability (collection to initiation of testing): Ambient: 72 hours; Refrigerated: 1 week; Frozen: Unacceptable

**New York State Clients:** Ambient: 24 hours; Refrigerated: Unacceptable; Frozen: Unacceptable