

Quarterly HOTLINE: Effective April 2, 2018

---

**New Test**      **3000472**      *Toxocara* Antibody by ELISA

**TOXOCA AB**

**Methodology:**      Semi-Quantitative Enzyme-Linked Immunosorbent Assay  
**Performed:**      Varies  
**Reported:**      3-11 days

**Specimen Required:** Collect: Plain Red or Serum Separator Tube (SST).  
Specimen Preparation: Transfer 1 mL serum to an ARUP Standard Transport Tube. (Min: 0.1 mL)  
Storage/Transport Temperature: Refrigerated. Also acceptable: Room temperature or frozen.  
Stability (collection to initiation of testing): Ambient: 1 week; Refrigerated: 2 weeks; Frozen: 1 month

**CPT Code(s):**      86682

New York DOH Approved.

**HOTLINE NOTE:** Refer to the Test Mix Addendum for interface build information.