

New Test **3000480** **Comprehensive Systemic Sclerosis Panel** **SCL COMPRE**



Additional Technical Information

Methodology: Qualitative Immunoblot/Semi-Quantitative Indirect Fluorescent Antibody/Semi-Quantitative Multiplex Bead Assay/Semi-Quantitative Enzyme-Linked Immunosorbent Assay
Performed: Thu
Reported: 1-8 days

Specimen Required: Collect: Serum Separator Tube (SST).
Specimen Preparation: Separate from cells ASAP or within 2 hours of collection. Transfer 3 mL serum to an ARUP Standard Transport Tube. (Min: 1.5 mL)
Storage/Transport Temperature: Refrigerated. Also acceptable: Room temperature or frozen.
Unacceptable Conditions: Hemolyzed, hyperlipemic, icteric, heat-treated or contaminated specimens
Stability (collection to initiation of testing): Ambient: 48 hours; Refrigerated: 2 weeks; Frozen: 1 year

Reference Interval:

Test Number	Components	Reference Interval	
0050599	Scleroderma (Scl-70) (ENA) Antibody, IgG	29 AU/mL or less	Negative
		30-40 AU/mL	Equivocal
		41 AU/mL or greater	Positive
0050470	RNP (U1) (Ribonucleic Protein) (ENA) Antibody, IgG	29 AU/mL or less	Negative
		30-40 AU/mL	Equivocal
		41 AU/mL or greater	Positive
3000082	Antinuclear Antibody (ANA) with HEp-2 Substrate, IgG by IFA	Less than 1:80	
2012173	Fibrillarin (U3 RNP) Antibody, IgG	Negative	
2003040	PM/Scl-100 Antibody, IgG by Immunoblot	Negative	
2001601	RNA Polymerase III Antibody, IgG	19 Units or less	Negative
		20-39 Units	Weak Positive
		40-80 Units	Moderate Positive
		81 Units or greater	Strong Positive

Interpretive Data: Refer to report

Note: Panel includes: Anti-Nuclear Ab (ANA) Titer, Anti-Nuclear Ab (ANA) Pattern, Anti-Scl-70, Anti-RNA Polymerase III Ab, Anti-U1 RNP Ab, Anti-Fibrillarin (U3 RNP), Anti-PM/Scl-100 Ab

CPT Code(s): 86039; 86235 x4; 83516

New York DOH Approved.

HOTLINE NOTE: Refer to the Test Mix Addendum for interface build information.