

Quarterly HOTLINE: Effective November 6, 2018

New Test Available Now	3001149	Uroplakin II by Immunohistochemistry	UROPII IHC
----------------------------------	----------------	---	-------------------

Methodology: Immunohistochemistry
Performed: Mon-Fri
Reported: 1-3 days

Specimen Required: Collect: Tissue.

Specimen Preparation: Formalin fix (10 percent neutral buffered formalin) and paraffin embed specimen (cells must be prepared into a cellblock). Protect paraffin block and/or slides from excessive heat. Transport tissue block or 5 unstained (3- to 5-micron thick sections), positively charged slides in a tissue transport kit (recommended but not required), (ARUP supply #47808) available online through eSupply using ARUP Connect or contact ARUP Client Services at (800) 522-2787. (Min: 2 slides) If sending precut slides, do not oven bake.

Storage/Transport Temperature: Room temperature. Also acceptable: Refrigerated. Ship in cooled container during summer months.

Unacceptable Conditions: Specimens submitted with non-representative tissue type. Depleted specimens.

Stability (collection to initiation of testing): Ambient: Indefinitely; Refrigerated: Indefinitely; Frozen: Unacceptable

Interpretive Data: See Compliance Statement B: www.aruplab.com/CS

Note: All stains will be handled as "Stain and Return" unless a consultation is requested. To request a consultation, submit the pathology report, all associated case materials (clinical history, blocks, slides, etc.), and the Anatomic Pathology requisition form (#32960) in place of the Immunohistochemistry Stain Form.

CPT Code(s): 88342

New York DOH approval pending. Call for status update.

HOTLINE NOTE: Refer to the Test Mix Addendum for interface build information.