

Quarterly HOTLINE: Effective February 19, 2019

---

**New Test**      **3001291**      *Neisseria gonorrhoeae* Antibody by CF, Serum      **N GONOR AB**

**Methodology:**      Semi-Quantitative Complement Fixation

**Performed:**      Varies

**Reported:**      3-7 days

**Specimen Required:** Collect: Plain Red or Serum Separator Tube (SST).

Specimen Preparation: Transfer 1 mL serum to an ARUP Standard Transport Tube. (Min: 0.5 mL)

Storage/Transport Temperature: Refrigerated. Also acceptable: Room temperature or frozen.

Stability (collection to initiation of testing): Ambient: 1 week; Refrigerated: 2 weeks; Frozen: 1 month

**CPT Code(s):**      86609

New York DOH Approved.

**HOTLINE NOTE:** Refer to the Test Mix Addendum for interface build information.