
0020040 Triglycerides, Serum or Plasma

TRG

Specimen Required: Patient Prep: Fasting specimen is preferred.

Collect: Plasma separator tube or serum separator tube.

Specimen Preparation: Allow specimen to clot completely at room temperature. Separate serum or plasma from cells ASAP or within 2 hours of collection. Transfer 1 mL serum or plasma to an ARUP Standard Transport Tube. (Min: 0.2 mL)

Storage/Transport Temperature: Refrigerated.

Unacceptable Conditions: Body Fluid (refer to Triglycerides, Fluid, ARUP test code 0020713). Collection tubes with glycerol-lubricated stoppers.

Stability (collection to initiation of testing): After separation from cells: Ambient: 48 hours; Refrigerated: 1 week; Frozen: 3 months

Reference Interval:

Effective February 22, 2022

By report