

TEST CHANGE

Zinc, Serum or Plasma

0020097, ZINC

Specimen Requirements:

Patient Preparation: Diet, medication, and nutritional supplements may introduce interfering substances. Upon the advice of their physician, patients should be encouraged to discontinue nutritional supplements, vitamins, minerals, and nonessential over-the-counter medications for one week prior to sample draw.

Collect: Royal blue (~~no additive~~), ~~royal~~ ~~No Additive~~, ~~Royal~~ blue (K2EDTA), or ~~r~~ ~~Royal~~ blue (NaHep).

Specimen Preparation: Separate from cells ASAP or within 2 hours of collection. Transfer 2 mL serum or plasma to an ARUP Trace Element-Free Transport Tube (ARUP supply #43116) available online through eSupply using ARUP ~~Connect(TM)~~ or ~~Connector~~ contact ARUP Client Services at ~~(800-)~~522-2787- (Min: 0.5 mL). Do not use utensils (i.e., syringes, needles, or pipettes) in the collection or transfer of the sample, pour directly into transport tube.

Transport Temperature: Room temperature. Also acceptable: Refrigerated or frozen.

Unacceptable Conditions: Specimens that are not separated from the red cells or clot within 2 hours. Specimens collected in containers other than specified. Specimens transported in containers other than specified. Hemolyzed specimens.

Remarks:

Stability: Ambient: Indefinitely; Refrigerated: Indefinitely; Frozen: Indefinitely

Methodology: Quantitative Inductively Coupled Plasma-Mass Spectrometry

Performed: Sun-Sat

Reported: 1-3 days

Note:

CPT Codes: 84630

New York DOH Approval Status: This test is New York DOH approved.

Interpretive Data:

Elevated results may be due to skin or collection-related contamination, including the use of a noncertified metal-free collection/transport tube. If contamination concerns exist due to elevated levels of serum/plasma zinc, confirmation with a second specimen collected in a certified metal-free tube is recommended.

Circulating zinc concentrations are dependent on albumin status and are depressed with malnutrition. Zinc may also be lowered with infection, inflammation, stress, oral contraceptives, and pregnancy. Zinc may be elevated with zinc supplementation or fasting. Elevated zinc concentrations may interfere with copper absorption.

~~This test was developed and its performance characteristics determined by ARUP Laboratories. It has not been cleared or approved by the US Food and Drug Administration. This test was performed in a CLIA-certified laboratory and is intended for clinical purposes.~~

Reference Interval:

60.0-120.0 µg/dL