

HOTLINE: Effective November 16, 2020

0020222 Hemosiderin, Urine**UHEMSID**

Methodology: Qualitative Microscopy
Performed: Sun-Sat
Reported: 1-3 days

Specimen Required: Patient Prep: First-morning collection is preferred.

Collect: Random urine.

Specimen Preparation: Mix specimen well. Transfer 4 mL to an ARUP Standard Transport Tube. (Min: 1 mL).

Storage/Transport Temperature: Frozen.

Unacceptable Conditions: Specimens in preservatives.

Stability (collection to initiation of testing): Ambient: 1 hour; Refrigerated: 1 week; Frozen: 1 week

Reference Interval:

Effective November 16, 2020

Absent

Interpretive Data:

As of Nov 16, 2020, reporting has switched from semi-quantitative to qualitative (absent versus present).

Note:

Absent = Negative

Present = 1+ to 4+