

0020454

Hepatitis B Virus Panel, Chronic with Reflex to HBsAg **Confirmation**

HEPCHRONIC

Methodology: Qualitative **Chemiluminescent** Immunoassay
Performed: Sun-Sat
Reported: **Within 24 hours**

Specimen Required: Patient Prep: Refer to individual components.

Collect: Serum separator tube (SST).

Specimen Preparation: Separate serum from cells ASAP or within 2 hours of collection. Transfer 3.5 mL serum to an ARUP Standard Transport Tube. (Min: 2.5 mL).

Storage/Transport Temperature: Refrigerated.

Unacceptable Conditions: Heparinized plasma, **specimens that are heat-inactivated, grossly hemolyzed, grossly icteric, grossly lipemic or specimens containing particulate material.**

Stability (collection to initiation of testing): After separation from cells: Ambient: Unacceptable; Refrigerated: 7 days; Frozen: 30 days (avoid repeated freeze/thaw cycles)