

0020498

Electrolytes, Urine

ULYTE

Specimen Required: Collect: 24-hour or random urine. Refrigerate during collection.

Specimen Preparation: Do not adjust specimen pH. Transfer a 1 mL aliquot of urine from a well-mixed collection to an ARUP Standard Transport Tube. (Min: 0.5 mL) Record total volume and collection time interval on transport tube and test request form.

Storage/Transport Temperature: Refrigerated.

Stability (collection to initiation of testing): Ambient: 48 hours; Refrigerated: 2 weeks; Frozen: 1 months