

HOTLINE: Effective November 2, 2020

0020715 Lipase, Fluid LIP FL

Specimen Required: Collect: Biliary/Hepatic, Drain, Pancreatic, Pericardial, Peritoneal/Ascites, Pleural or Synovial fluid.

Specimen Preparation: Centrifuge to remove cellular material. Transfer 1 mL body fluid to an ARUP Standard Transport Tube. (Min:

0.5 mL)

<u>Storage/Transport Temperature:</u> Room temperature. <u>Remarks:</u> **Specimen source must be provided.**

<u>Unacceptable Conditions:</u> Specimen types other than those listed. Specimens too viscous to be aspirated by instrument.

Stability (collection to initiation of testing): Ambient: 1 week; Refrigerated: 1 week; Frozen: 2 months