
0020849

Potassium, Urine

U K

Specimen Required: Collect: 24-hour or random urine without additives. Refrigerate during collection.

Specimen Preparation: Transfer a 1 mL aliquot of urine from a well-mixed collection to an ARUP Standard Transport Tube. (Min: 0.2 mL) Record total volume and collection time interval on transport tube and test request form.

Storage/Transport Temperature: Refrigerated.

Stability (collection to initiation of testing): Ambient: 14 Days; Refrigerated: 2 weeks; Frozen: 1 month