

HOTLINE: Effective May 20, 2019

0025055 Heavy Metals Panel 6, Urine with Reflex to Arsenic Fractionated

HYMET 6

Performed:Sun-SatReported:1-5 days

Specimen Required: Patient Prep: Diet, medication, and nutritional supplements may introduce interfering substances. Patients should be encouraged to discontinue nutritional supplements, vitamins, minerals, non-essential over-the-counter medications (upon the advice of their physician), and avoid shellfish and seafood for 48 to 72 hours. Collection from patients receiving iodinated or gadolinium-based contrast media must be avoided for a minimum of 72 hours post-exposure. Collection from patients with impaired kidney function should be avoided for a minimum of 14 days post-contrast media exposure.
Collect: 24 Hour Urine. Refrigerate during collection. Specimen must be collected in a plastic container. Also acceptable: Random Urine.
Specimen Preparation: Transfer 8 mL aliquot from a well-mixed collection to ARUP Trace Element-Free Transport Tubes (ARUP)

Specimen Preparation: Transfer 8 mL aliquot from a well-mixed collection to ARUP Trace Element-Free Transport Tubes (ARUP supply #43116). Available online through eSupply using ARUP ConnectTM or contact ARUP Client Services at (800) 522-2787. (Min: 2 mL)

Storage/Transport Temperature: Refrigerated. Also acceptable: Room temperature or frozen.

<u>Remarks:</u> Trace Elements requisition form may be required (ARUP form #32990-Barcode; #32991-No Barcode). Record total volume and collection time interval on transport tube and on test request form.

<u>Unacceptable Conditions:</u> Specimens collected within 72 hours after administration of iodinated or gadolinium-based contrast media. Acid preserved urine. Specimens transported in containers other than specified. Specimens contaminated with blood or fecal material. Stability (collection to initiation of testing): Ambient: 1 week; Refrigerated: 2 weeks; Frozen: 1 year

Note: High concentrations of iodine or gadolinium may interfere with elemental testing. If total arsenic concentration is between 35-2000 ug/L, then Arsenic, Fractionated, will be added to determine the proportion of organic, inorganic, and methylated forms. Additional charges apply.