

## **TEST CHANGE**

Fibrinogen Antigen   0030135, FIBAG   Specimen Requirements:   Patient Preparation:   Collect: Lt. blue (sodium citrate). Refer to Specimen Handling at aruplab.com for hemostasis/thrombosis specimen handling guidelines.   Specimen Preparation: Transfer 2 mL platelet-poor plasma to an ARUP Standard Transport Tube. (Min: 1 mL)   Transport Temperature: CRITICAL FROZEN. Separate specimens must be submitted when multiple tests are ordered.   Unacceptable Conditions: Serum. EDTA plasma, clotted, grossly lipemic, or hemolyzed specimens.   Remarks: Stability:   Stability: Ambient: 8 hours; Refrigerated: Unacceptable; Frozen: 1 month   Methodology: Quantitative Immunoturbidimetry   Performed: Tue, Fri   Reported: 2-4 days   Note: CPT Codes:   CPT Codes: 85385   New York DOH Approval Status: This test is New York DOH approved.   Interpretive Data: This test is New York DOH approved.   Interpretive Data: This test is New York DOH approved.		
Specimen Requirements:   Patient Preparation:   Collect: Lt. blue (sodium citrate). Refer to Specimen Handling at aruplab.com for hemostasis/thrombosis specimen handling guidelines.   Specimen Preparation: Transfer 2 mL platelet-poor plasma to an ARUP Standard Transport Tube. (Min: 1 mL)   Transport Temperature: CRITICAL FROZEN. Separate specimens must be submitted when multiple tests are ordered.   Unacceptable Conditions: Serum. EDTA plasma, clotted, grossly lipemic, or hemolyzed specimens.   Remarks: Ambient: 8 hours; Refrigerated: Unacceptable; Frozen: 1 month   Methodology: Quantitative Immunoturbidimetry   Performed: Tue, Fri   Reported: 2-4 days   Note: Stability:   OPT Codes: 85385   New York DOH Approval Status: This test is New York DOH approved.   Interpretive Data: Reference Interval:	Fibrinogen Antigen	
Patient Preparation:Collect:Lt. blue (sodium citrate). Refer to Specimen Handling at aruplab.com for hemostasis/thrombosis specimen handling guidelines.Specimen Preparation:Transfer 2 mL platelet-poor plasma to an ARUP Standard Transport Tube. (Min: 1 mL)Transport Temperature:CRITICAL FROZEN. Separate specimens must be submitted when multiple tests are ordered.Unacceptable Conditions:Serum. EDTA plasma, clotted, grossly lipemic, or hemolyzed specimens.Remarks:Serum. EDTA plasma, clotted, grossly lipemic, or hemolyzed specimens.Methodology:Quantitative ImmunoturbidimetryPerformed:Tue, FriReported:2-4 daysNote:SassCPT Codes:85385New York DOH Approval Status:This test is New York DOH approved.Interpretive Data:This test is New York DOH approved.Reference Interval:Unaccentraval	·	
Collect:Lt. blue (sodium citrate). Refer to Specimen Handling at aruplab.com for hemostasis/thrombosis specimen handling guidelines.Specimen Preparation:Transfer 2 mL platelet-poor plasma to an ARUP Standard Transport Tube. (Min: 1 mL)Transport Temperature:CRITICAL FROZEN. Separate specimens must be submitted when multiple tests are ordered.Unacceptable Conditions:Serum. EDTA plasma, clotted, grossly lipemic, or hemolyzed specimens.Remarks:Serum. EDTA plasma, clotted, grossly lipemic, or hemolyzed specimens.Methodology:Quantitative ImmunoturbidimetryPerformed:Tue, FriReported:2-4 daysNote:Stability:CPT Codes:85385New York DOH Approval Status:This test is New York DOH approved.Interpretive Data:This test is New York DOH approved.Reference Interval:Unaccenterval:	Specimen Requirements:	
aruplab.com for hemostasis/thrombosis specimen handling guidelines.Specimen Preparation:Transfer 2 mL platelet-poor plasma to an ARUP Standard Transport Tube. (Min: 1 mL)Transport Temperature:CRITICAL FROZEN. Separate specimens must be submitted when multiple tests are ordered.Unacceptable Conditions:Serum. EDTA plasma, clotted, grossly lipemic, or hemolyzed specimens.Remarks:Serum. EDTA plasma, clotted, grossly lipemic, or hemolyzed specimens.Nethodology:Quantitative ImmunoturbidimetryPerformed:Tue, FriReported:2-4 daysNote:Stability:CPT Codes:85385New York DOH Approval Status:This test is New York DOH approved.Interpretive Data:This test is New York DOH approved.	Patient Preparation:	
Transport Tube. (Min: 1 mL)Transport Temperature:CRITICAL FROZEN. Separate specimens must be submitted when multiple tests are ordered.Unacceptable Conditions:Serum. EDTA plasma, clotted, grossly lipemic, or hemolyzed specimens.Remarks:Ambient: 8 hours; Refrigerated: Unacceptable; Frozen: 1 monthMethodology:Quantitative ImmunoturbidimetryPerformed:Tue, FriReported:2-4 daysNote:CPT Codes:CPT Codes:85385New York DOH Approval Status:This test is New York DOH approved.Interpretive Data:Circumanian CompanyReformed:This test is New York DOH approved.	Collect:	aruplab.com for hemostasis/thrombosis specimen handling
when multiple tests are ordered.Unacceptable Conditions:Serum. EDTA plasma, clotted, grossly lipemic, or hemolyzed specimens.Remarks:Stability:Stability:Ambient: 8 hours; Refrigerated: Unacceptable; Frozen: 1 monthMethodology:Quantitative ImmunoturbidimetryPerformed:Tue, FriReported:2-4 daysNote:CPT Codes:CPT Codes:85385New York DOH Approval Status:This test is New York DOH approved.Interpretive Data:	Specimen Preparation:	
Remarks:Stability:Ambient: 8 hours; Refrigerated: Unacceptable; Frozen: 1 monthMethodology:Quantitative ImmunoturbidimetryPerformed:Tue, FriReported:2-4 daysNote:2-4 daysCPT Codes:85385New York DOH Approval Status:This test is New York DOH approved.Interpretive Data:This test is New York DOH approved.Reference Interval:Status Status	Transport Temperature:	
Stability:Ambient: 8 hours; Refrigerated: Unacceptable; Frozen: 1 monthMethodology:Quantitative ImmunoturbidimetryPerformed:Tue, FriReported:2-4 daysNote:2-4 daysCPT Codes:85385New York DOH Approval Status:This test is New York DOH approved.Interpretive Data:	Unacceptable Conditions:	
Methodology:Quantitative ImmunoturbidimetryPerformed:Tue, FriReported:2-4 daysNote:2-4 daysCPT Codes:85385New York DOH Approval Status:This test is New York DOH approved.Interpretive Data:This test is New York DOH approved.Reference Interval:Status Status	Remarks:	
Performed:Tue, FriReported:2-4 daysNote:CPT Codes:85385New York DOH Approval Status:This test is New York DOH approved.Interpretive Data:Reference Interval:	Stability:	Ambient: 8 hours; Refrigerated: Unacceptable; Frozen: 1 month
Reported:2-4 daysNote:CPT Codes:85385New York DOH Approval Status:This test is New York DOH approved.Interpretive Data:Reference Interval:	Methodology:	Quantitative Immunoturbidimetry
Note: CPT Codes: 85385   New York DOH Approval Status: This test is New York DOH approved.   Interpretive Data: Feference Interval:	Performed:	Tue, Fri
CPT Codes:85385New York DOH Approval Status:This test is New York DOH approved.Interpretive Data:Interpretive Data:Reference Interval:Interval:	Reported:	2-4 days
New York DOH Approval Status: This test is New York DOH approved.   Interpretive Data: Reference Interval:	Note:	
Interpretive Data: Reference Interval:	CPT Codes:	85385
Reference Interval:	New York DOH Approval Status:	This test is New York DOH approved.
	Interpretive Data:	
149-353 mg/dL	Reference Interval:	
	149-353 mg/dL	

HOTLINE NOTE: There is a numeric map change associated with this test. Refer to the Hotline Test Mix for interface build information.