

---

**0050002****Alpha-1-Acid Glycoprotein****A1A GLYCO**

**Specimen Required:** Collect: Serum separator tube or green (sodium or lithium heparin).

Specimen Preparation: Separate serum or plasma from cells ASAP or within 2 hours of collection. Transfer 1 mL serum or plasma to an ARUP Standard Transport Tube. (Min: 0.5 mL)

Storage/Transport Temperature: Refrigerated.

Unacceptable Conditions: Hemolyzed specimens.

Stability (collection to initiation of testing): After separation from cells: Ambient: 8 hours; Refrigerated: 8 days; Frozen: 3 months