

---

**0050135      *Brucella* Antibody (Total) by Agglutination**

**BRUC**

**Specimen Required:** Collect: Serum Separator Tube (SST).

Specimen Preparation: Separate serum from cells ASAP or within 2 hours of collection. Transfer 1 mL serum to an ARUP Standard Transport Tube. (Min: 0.3 mL) Parallel testing is preferred and convalescent specimens **must** be received within 30 days from receipt of the acute specimens.

Storage/Transport Temperature: Refrigerated.

Remarks: **Mark specimens plainly as "acute" or "convalescent."**

Unacceptable Conditions: Contaminated, heat-inactivated, hemolyzed, or severely lipemic specimens.

Stability (collection to initiation of testing): After separation from cells: Ambient: 48 hours; Refrigerated: 2 weeks; Frozen: 6 months (avoid repeated freeze/thaw cycles)