

---

**0050172*****Blastomyces dermatitidis* Antibodies by Immunodiffusion****BLASTO PPT****Performed:** Sun-Fri  
**Reported:** 3-6 days**Specimen Required:** Collect: Serum separator tube.Specimen Preparation: Separate serum from cells ASAP or within 2 hours of collection. Transfer 0.5 mL serum to an ARUP Standard Transport Tube. (Min: 0.15 mL)Storage/Transport Temperature: Refrigerated.Unacceptable Conditions: Body fluids.Stability (collection to initiation of testing): After separation from cells: Ambient: 48 hours; Refrigerated: 2 weeks; Frozen: 1 year (avoid repeated freeze/thaw cycles)