
0050174***Histoplasma* spp. Antibodies by Immunodiffusion****HISTO PPT**

Performed: Sun-Fri
Reported: 3-6 days

Specimen Required: Collect: Serum separator tube.

Specimen Preparation: Separate serum from cells ASAP or within 2 hours of collection. Transfer 0.5 mL serum to an ARUP Standard Transport Tube. (Min 0.15 mL)

Storage/Transport Temperature: Refrigerated.

Unacceptable Conditions: Body fluids.

Stability (collection to initiation of testing): After separation from cells: Ambient: 48 hours; Refrigerated: 2 weeks; Frozen: 1 year (avoid repeated freeze/thaw cycles)