

HOTLINE: Effective February 16, 2021

0050631 Immunoglobulins, CSF Quantitative

QNTIG CSF

Methodology: Quantitative Immunoturbidimetry

Specimen Required: Collect: CSF.

Specimen Preparation: Centrifuge and separate to remove cellular material. Transfer 1 mL CSF to an ARUP Standard Transport Tube. (Min: 0.6 mL)

Storage/Transport Temperature: Refrigerated.

Unacceptable Conditions: Grossly bloody or hemolyzed specimens.

Stability (collection to initiation of testing): Ambient: Unacceptable; Refrigerated: 1 month; Frozen: 6 months

Reference Interval:

Test Number	Components	Reference Interval
0050341	Immunoglobulin A, CSF	0.0-0.7 mg/dL
3003485	Immunoglobulin G, CSF	0.0-6.0 mg/dL
0050356	Immunoglobulin M, CSF	0.0-0.7 mg/dL