

0051003

*Ehrlichia chaffeensis* Antibody, IgM by IFA

E CH M

**Specimen Required:** Collect: Serum Separator Tube.Specimen Preparation: Separate from cells ASAP or within 2 hours of collection. Transfer 1 mL serum to an ARUP Standard Transport Tube. (Min: 0.3 mL) Parallel testing is preferred and convalescent specimens **must** be received within 30 days from receipt of the acute specimens. **Mark specimens plainly as acute or convalescent.**Storage/Transport Temperature: Refrigerated.Unacceptable Conditions: Contaminated, hemolyzed, or severely lipemic specimens.Stability (collection to initiation of testing): After separation from cells: Ambient: 48 hours; Refrigerated: 2 weeks; Frozen: 1 year (avoid repeated freeze/thaw cycles)