

Quarterly HOTLINE: Effective February 19, 2019

0051004

Ehrlichia chaffeensis Antibody, IgG by IFA

E CH G

Specimen Required: Collect: Serum separator tube.

Specimen Preparation: Separate serum from cells ASAP or within 2 hours of collection. Transfer 1 mL serum to an ARUP Standard Transport Tube. (Min: 0.1 mL) Parallel testing is preferred and convalescent specimens **must** be received within 30 days from receipt of the acute specimens. **Mark specimens plainly as acute or convalescent.**

Storage/Transport Temperature: Refrigerated.

Unacceptable Conditions: Contaminated, hemolyzed, or severely lipemic specimens.

Stability (collection to initiation of testing): After separation from cells: Ambient: 48 hours; Refrigerated: 2 weeks; Frozen: 1 year (avoid repeated freeze/thaw cycles)