

HOTLINE: Effective August 16, 2021

0055167

Kappa/Lambda Quantitative Free Light Chains with Ratio, Serum

KAP/LAM F

Specimen Required: Collect: Serum Separator Tube (SST).

Specimen Preparation: Separate from cells ASAP or within 2 hours of collection. Transfer 2 mL serum to an ARUP Standard

Transport Tube: (Min: 0.5 mL)

Storage/Transport Temperature: Refrigerated.

Unacceptable Conditions: Plasma. **Grossly Hemolyzed and Lipemic Samples**

Stability (collection to initiation of testing): After separation from cells: Ambient: Unacceptable; Refrigerated: 3 weeks; Frozen: 6 months