

HOTLINE: Effective May 20, 2019

0058002	<i>Campylobacter</i> Antigen	CAMPY
Methodology:	Qualitative Enzyme Immunoassay	
Performed:	Sun-Sat	
Reported:	1-2 days	
Specimen Required:	<u>Collect:</u> Stool. <u>Specimen Preparation:</u> Transport 5 g stool to an unpreserved stool transport vial (ARUP supply #40910) available online through eSupply using ARUP Connect™ or contact ARUP Client Services at (800) 522-2787. (Min: 1 g) Also acceptable: Transfer 5 g stool within one hour of collection to enteric transport media (Cary-Blair) (ARUP supply #29799) available online through eSupply using ARUP Connect™ or contact ARUP Client Services at (800) 522-2787. (Min: 1 g) <u>Storage/Transport Temperature:</u> Refrigerated. <u>Unacceptable Conditions:</u> Specimens in any transport media other than indicated above. <u>Stability (collection to initiation of testing):</u> <b>Unpreserved Stool:</b> Ambient: Unacceptable; Refrigerated: 4 days; Frozen: 1 week <b>Cary-Blair/C&amp;S Media:</b> Ambient: 4 days; Refrigerated: 4 days; Frozen: Unacceptable	
CPT Code(s):	87449	