

TEST CHANGE

Lactoferrin, Fecal by ELISA

0061164, FECLACTO

Specimen Requirements:

Patient Preparation:

Collect: Stool.

Specimen Preparation: Transfer 5 g stool to an unpreserved stool transport vial (ARUP supply #40910). Available online through eSupply using ARUP Connect(TM) or contact Client Services at [800-522-2787](tel:800-522-2787). (Min: [1 g](tel:800-522-2787))([800-522-2787](tel:800-522-2787). (Min: [1 g](tel:800-522-2787)) Also acceptable: Place 5 g stool in enteric transport media (Cary-Blair) (ARUP Supply #29799). Available online through eSupply using ARUP Connect(TM) or contact ARUP Client Services at [\(800\) 522-2787](tel:800-522-2787).

Transport Temperature: Refrigerated.

Unacceptable Conditions: Specimens in any transport media ~~than indicated above~~.

Remarks:

Stability: ~~Unpreserved:~~ Ambient: 2 weeks; Refrigerated: 2 weeks; Frozen: 2 weeks
~~Preserved:~~ Ambient: 1 week; Refrigerated: 2 weeks; Frozen: 2 weeks

Methodology: Qualitative Enzyme-Linked Immunosorbent Assay ([ELISA](#))

Note:

CPT Codes: 83630

New York DOH Approval Status: This test is New York DOH approved.

Interpretive Data:

A positive result is indicative of the presence of lactoferrin, a marker for fecal leukocytes. A negative result does not exclude the presence of intestinal inflammation.

Reference Interval:

Negative

Test Number	Components	Reference Interval
	Lactoferrin, Fecal by ELISA	Negative