
0070037

Histamine, Whole Blood

HIST-WB

Specimen Required: Collect: Green (sodium or lithium heparin).

Specimen Preparation: Transfer 1 mL well-mixed whole blood to an ARUP Standard Transport Tube and freeze. (Min: 0.5 mL)

Storage/Transport Temperature: **CRITICAL FROZEN. Separate specimens must be submitted when multiple tests are ordered.**

Stability (collection to initiation of testing): Ambient: **Unacceptable**; Refrigerated: **Unacceptable**; Frozen: 6 months

HOTLINE NOTE: Remove information found in the Specimen Required Unacceptable Conditions field.