

---

**0070038****Histamine, Urine****HIST-U**

**Specimen Required:** Collect: Random or 24-hour urine in a plastic container. Refrigerate during collection.

Specimen Preparation: Transfer a 4 mL aliquot from a well-mixed random or 24-hour collection to an ARUP Standard Transport Tube and freeze immediately. (Min: 1 mL) Record total volume and collection time interval on transport tube and test request form.

Storage/Transport Temperature: **CRITICAL FROZEN. Separate specimens must be submitted when multiple tests are ordered.**

Stability (collection to initiation of testing): Ambient: Unacceptable; Refrigerated: 6 hours; Frozen: 6 months

**HOTLINE NOTE:** Remove information found in the Specimen Required Unacceptable Conditions field.