

HOTLINE: Effective May 10, 2021

0070045

Estradiol, Adult Premenopausal Female, Serum or Plasma

ESTRA

Specimen Required: Collect: Serum Separator Tube (SST). Also acceptable: Green (Sodium or Lithium Heparin)

Specimen Preparation: Separate from cells ASAP or within 2 hours of collection. Transfer 1 mL serum or plasma to an ARUP

Standard Transport Tube. (Min: 0.4 mL)

Storage/Transport Temperature: Frozen.

Unacceptable Conditions: Grossly hemolyzed or lipemic specimens.

Stability (collection to initiation of testing): After separation from cells: Ambient: 8 hours; Refrigerated: 1 week; Frozen: 6 months

Reference Interval:

Effective May 10, 2021

Female	
Follicular phase	27-122 pg/mL
Mid Cycle phase	95-433 pg/mL
Luteal Phase	49-291 pg/mL
Post-Menopausal	Less than 41 pg/mL

HOTLINE NOTE: Remove information found in the Interpretive Data field.