
0070053**Bone Specific Alkaline Phosphatase****BSAP**

Specimen Required: Collect: Serum separator tube. Also acceptable: Green (sodium or lithium heparin).

Specimen Preparation: Transfer 0.5 mL serum or plasma to an ARUP Standard Transport Tube. (Min: 0.3 mL)

Storage/Transport Temperature: **CRITICAL FROZEN. Separate specimens must be submitted when multiple tests are ordered.**

Unacceptable Conditions: Urine. Grossly hemolyzed specimens.

Stability (collection to initiation of testing): After separation from cells: Ambient: 2 hours; Refrigerated: 48 hours; Frozen: 2 months