

---

**0070066**

**Insulin, 60 Minutes**

**INSULIN 60**

**Specimen Required:** Collect: Serum separator tube.

Specimen Preparation: Allow specimen to clot completely at room temperature. Separate serum from cells ASAP or within 2 hours of collection. Transfer 1 mL serum to an ARUP Standard Transport Tube. (Min: 0.4 mL)

Storage/Transport Temperature: Frozen.

Unacceptable Conditions: Heparinized plasma. I.V. fluid. Vitreous fluid. Gray (sodium fluoride/potassium oxalate). Hemolyzed specimens.

Stability (collection to initiation of testing): After separation from cells: Ambient: 8 hours; Refrigerated: 1 week; Frozen: 1 month