

TEST CHANGE

Proinsulin, Intact

0070112, PROINS

Specimen Requirements:

Patient Preparation:	Patient must fast for 10-12-15 hours prior to collection.
Collect:	Serum separator tube (SST) or plain red. Also acceptable: Lavender (K2EDTA) or pink (K2EDTA).
Specimen Preparation:	Separate from cells ASAP or within 2 hours of collection. Transfer 1 mL serum or plasma to an ARUP standard transport tube and freeze immediately. (Min: 0.2 mL)
Transport Temperature:	CRITICAL FROZEN. Separate specimens must be submitted when multiple tests are ordered.
Unacceptable Conditions:	Grossly hemolyzed specimens.

Remarks:

Stability:	After separation from cells: Ambient: Unacceptable; Refrigerated: 24 hours; Frozen: 2 months
------------	---

Methodology: Quantitative Chemiluminescent Immunoassay (CLIA)

Note:

CPT Codes: 84206

New York DOH Approval Status: This test is New York DOH approved.

Interpretive Data:

Reference Interval:

Test Number	Components	Reference Interval	
	Proinsulin, Intact		
		Age	pmol/L
		0-17 year	Not established
		18 years and older	Less than or equal to 7.3