

HOTLINE: Effective February 22, 2022

0070193

Luteinizing Hormone and Follicle Stimulating Hormone

LH/FSH

Specimen Required: Collect: Serum separator tube, plasma separator tube, Green (lithium heparin), Lavender (K₂ EDTA or K₃ EDTA).
Specimen Preparation: Allow specimen to clot completely at room temperature. Separate serum or plasma from cells ASAP or within 2 hours of collection. Transfer 2 mL serum or plasma to an ARUP Standard Transport Tube. (Min: 0.5 mL)
Storage/Transport Temperature: Refrigerated.
Unacceptable Conditions: Grossly hemolyzed specimens.
Stability (collection to initiation of testing): After separation from cells: Ambient: 5 days; Refrigerated: 2 weeks; Frozen: 6 months