

---

**0070283****Soluble Transferrin Receptor****STR**

**Specimen Required:** Collect: Serum separator tube or plasma separator tube. Also acceptable: **green (lithium heparin)**.

Specimen Preparation: Transfer 1 mL serum or plasma to an ARUP Standard Transport Tube. (Min: 0.3 mL)

Storage/Transport Temperature: Refrigerated.

Unacceptable Conditions: Contaminated, severely hemolyzed, icteric, or lipemic specimens.

Stability (collection to initiation of testing): After separation from cells: Ambient: 72 hours; Refrigerated: 1 week; Frozen: 1 month (avoid repeated freeze/thaw cycles)