
0080397**Serotonin, Serum****SEROT-SER**

Specimen Required: Patient Prep: Abstain from medications for 72 hours prior to collection.

Collect: Serum Separator Tube (SST).

Specimen Preparation: Separate from cells within 1 hour of collection. Transfer 0.5 mL serum to an ARUP Amber Transport Tube. (Min: 0.2 mL)

Storage/Transport Temperature: Frozen. Separate specimens must be submitted when multiple tests are ordered.

Unacceptable Conditions: Specimens other than serum. Non-frozen specimens.

Stability (collection to initiation of testing): After separation from cells: Ambient: Unacceptable; Refrigerated: 24 hours; Frozen: 1 month