

TEST CHANGE

Cocaine Metabolite, Urine, Quantitative

0090359, CDCO COCA

Specimen Requirements:

Patient Preparation:

Collect: Random urine.

Specimen Preparation: Transfer 1 mL urine with no additives or preservatives to an ARUP standard transport tube. (Min: 0.5 mL)

Transport Temperature: Room temperature.

Unacceptable Conditions: Specimens exposed to repeated freeze/thaw cycles.

Remarks:

Stability: Ambient: 1 week; Refrigerated: 1 month; Frozen: 3 years

Methodology: Quantitative Liquid Chromatography-Tandem Mass Spectrometry

Performed: Sun-Sat

Reported: 1-~~6~~4 days

Note: Compare to Pain Management, Cocaine Metabolite with Confirmation with medMATCH, Urine; Pain Management, Cocaine Metabolite, Quantitative, with medMATCH, Urine.

CPT Codes: 80353 (Alt code: G0480)

New York DOH Approval Status: This test is New York DOH approved.

Interpretive Data:

Methodology: Quantitative Liquid Chromatography-Tandem Mass Spectrometry

Positive cutoff: 50 ng/mL

For medical purposes only; not valid for forensic use.

The concentration value must be greater than or equal to the cutoff to be reported as positive. Interpretive questions should be directed to the laboratory.

Reference Interval:

Effective August 17, 2015

Drugs Covered	Cutoff Concentrations	
Benzoylcegonine	50 ng/mL	