

TEST CHANGE

Flunitrazepam and Metabolites, Serum or Plasma, Screen with Reflex to Confirmation/Quantitation

0091116, FLUNITR SP

Specimen Requirements:

Patient Preparation:

Collect: Plain red, lavender (EDTA), or pink (K2EDTA).

Specimen Preparation: Separate from cells ASAP or within 2 hours of collection. Transfer 3 mL serum or plasma to an ARUP standard transport tube~~Standard Transport Tube~~. (Min: 1.4 mL)
Test is not performed at ARUP; separate specimens must be submitted when multiple tests are ordered.

Transport Temperature: ~~Frozen~~Refrigerated. Also acceptable: ~~Refrigerated~~Frozen.

Unacceptable Conditions: Polymer gel separation tube (SST or PST)~~Separator tubes~~.

Remarks:

Stability: Ambient: 48 hours; Refrigerated: 2 weeks; Frozen: 3 months

Methodology: Qualitative High Performance Liquid Chromatography-Tandem Mass Spectrometry

Note: If screen is positive, then confirmation will be added. Additional charges apply.

CPT Codes: 80307; if reflexed, add 80346 (~~Reflexed Alt Code: G0480~~)

New York DOH Approval Status: This test is New York DOH approved.

Interpretive Data:

Reference Interval:

Refer to~~By~~ report