
0091203

Heroin - Screen with Reflex to Confirmation/Quantitation - Serum or Plasma

HEROIN SP

Specimen Required: Collect: Plain **Red or Gray** (Sodium Fluoride/Potassium **Oxalate**).

Specimen Preparation: Separate from cells ASAP or within 2 hours of collection. Transfer 3 mL serum or plasma to an ARUP Standard Transport Tube. (Min: 1.3 mL)

Test is not performed at ARUP; separate specimens must be submitted when multiple tests are ordered.

Storage/Transport Temperature: Refrigerated. Also acceptable: Frozen.

Unacceptable Conditions: Separator tubes.

Stability (collection to initiation of testing): Ambient: 48 hours; Refrigerated: 2 weeks; Frozen: 1 month