
0091434**Titanium Quantitative, Serum or Plasma****TITANIUM SP**

Performed: Varies
Reported: 5-8 days

Specimen Required: Collect: Royal Blue (No Additive) or Royal Blue (Na₂ or K₂EDTA).

Specimen Preparation: **Separate from cells ASAP or within 2 hours of collection.** Transfer 2 mL serum or plasma to an Acid Washed Transfer Vial (ARUP supply #54350) available online through eSupply using ARUP Connect™ or contact ARUP Client Services at (800) 522-2787. (Min: 0.6 mL)

Storage/Transport Temperature: Refrigerated. Also acceptable: Room temperature or frozen.

Unacceptable Conditions: Separator tubes.

Stability (collection to initiation of testing): Ambient: 1 month; Refrigerated: 1 month; Frozen: 1 month