

---

**0092050**

**Human Immunodeficiency Virus (HIV) Phenotype Comprehensive**

**HIV COMPRE**

**Specimen Required:** Collect: Lavender (EDTA) or plasma preparation tube (PPT).

Specimen Preparation: Separate plasma from cells within 6 hours of collection. Transfer 3 mL plasma to an ARUP Standard Transport Tube and freeze immediately. (Min: 3 mL)

Storage/Transport Temperature: **CRITICAL FROZEN. Separate specimens must be submitted when multiple tests are ordered.**

Remarks: Provide patient's most recent viral load and viral load collection date. Procedure should be used for patients with documented HIV-1 infection and viral loads greater than 500 copies/mL.

Unacceptable Conditions: Thawed specimens.

Stability (collection to initiation of testing): Ambient: Unacceptable; Refrigerated: Unacceptable; **Frozen: 2 weeks**