

TEST CHANGE

17-Hydroxypregnenolone 30-Minute Timed Specimen

0092336, OHPRGN 30

Specimen Requirements:

Patient Preparation:

Collect: Serum separator tube. Also acceptable: Plain red, lavender (EDTA), pink (K2EDTA), or green (sodium or lithium heparin).

Specimen Preparation: Separate serum or plasma from cells ASAP or within 2 hours of collection. Transfer 0.5 mL serum or plasma to an ARUP Standard Transport Tube and freeze immediately. (Min: 0.25 mL)

Transport Temperature: CRITICAL FROZEN. Separate specimens must be submitted when multiple tests are ordered.

Unacceptable Conditions: Refrigerated or room temperature specimens.

Remarks:

Stability: After separation from cells: Ambient: Unacceptable; Refrigerated: Unacceptable; Frozen: 6 months

Methodology: Quantitative High Performance Liquid Chromatography-Tandem Mass Spectrometry

Performed: Mon-Fri

Reported: 1-54 days

Note:

CPT Codes: 84143

New York DOH Approval Status: This test is New York DOH approved.

Interpretive Data:

This test was developed and its performance characteristics determined by ARUP Laboratories. It has not been cleared or approved by the US Food and Drug Administration. This test was performed in a CLIA certified laboratory and is intended for clinical purposes.

Reference Interval:

By report