
0092399

HIV PhenoSense GT

HIVPHENOGT

Specimen Required: Collect: Lavender (EDTA) or Plasma Preparation Tube (PPT).

Specimen Preparation: Separate from cells within 6 hours of collection. Transfer 3 mL plasma to an ARUP Standard Transport Tube and freeze immediately. (Min: 1 mL)

Storage/Transport Temperature: **CRITICAL FROZEN. Separate specimens must be submitted when multiple tests are ordered.**

Remarks: Provide patient's most recent viral load and viral load collection date.

Unacceptable Conditions: Thawed specimens.

Stability (collection to initiation of testing): Ambient: Unacceptable; Refrigerated: Unacceptable; **Frozen: 2 weeks**