

TEST CHANGE

Arylsulfatase A, 24-Hour Urine

0095227, ARYLSULF A

Specimen Requirements:

Patient Preparation:

Collect: 24-hour urine. Do not use preservatives. Keep refrigerated during collection.

Specimen Preparation: Transfer 6 mL urine to ARUP ~~standard transport tubes~~ **Standard Transport Tubes**. (Min. 2.5 mL) Test is not performed at ARUP; separate specimens must be submitted when multiple tests are ordered.

Transport Temperature: Refrigerated.

Unacceptable Conditions: Random urine.

Remarks: Indicate total volume.

Stability: Ambient: Unacceptable; Refrigerated: 2 weeks; Frozen: Unacceptable

Methodology: Quantitative Colorimetry/Enzyme Immunoassay **(EIA)**

Performed: Varies

Reported: 9-~~19~~**17** days

Note:

CPT Codes: 84311

New York DOH Approval Status: This test is New York DOH approved.

Interpretive Data:

Reference Interval:

By report