

TEST CHANGE

Allergen, Tree, Oak Red IgE

0097933, OAK RED

Specimen Requirements:	
Patient Preparation:	N/A
Collect:	Plain red or serum separator tube (SST).
Specimen Preparation:	Separate serum from cells ASAP or within 2 hours of collection. Transfer 0.5 mL serum plus 0.1 mL for each additional allergen ordered to an ARUP standard transport tube. Standard Transport Tube. (Min: 0.34 mL plus 0.04 mL for each allergen ordered) Test is not performed at ARUP; separate specimens must be submitted when multiple tests are ordered.
Transport Temperature:	Frozen. Room temperature. Also acceptable: Room temperature Refrigerated or refrigerated frozen.
Unacceptable Conditions:	Hemolyzed, icteric, or lipemic specimens.
Remarks:	
Stability:	Ambient: 1 month; Refrigerated: 1 month; Frozen: 1 year
Methodology:	Quantitative Enzyme Immunoassay (EIA)
Note:	
CPT Codes:	86003
New York DOH Approval Status:	This test is New York DOH approved.
Interpretive Data:	
N/A	
Reference Interval:	
Refer to By report	

Deleted Cells
Deleted Cells