

TEST CHANGE

17-Hydroxyprogesterone, Urine

0098378. 170HPROGU

Specimen Requirements: Patient Preparation: Patient should not be on any corticosteroid, ACTH, estrogen, or gonadotropin medications, if possible, for at least 48 hours prior to collection of specimen. Collect: 24-hour urine. No special preservatives required. Specimen Preparation: Mix specimen well. Refrigerate during collection. Transfer 10 mL aliquot of urine to ARUP standard transport tubes Standard Transport Temperature: Prozen. On dry ice is preferred. Separate specimens must be submitted when multiple tests are ordered. Unacceptable Conditions: Remarks: Stability: Ambient: 1 hour; Refrigerated: 4 days; Frozen: 6 months Methodology: Quantitative Radioimmunoassay. (RIA). Performed: Varies Reported: 7-1340 days Note: CPT Codes: 83498 New York DOH Approval Status: Specimens from New York clients will be sent out to a New York DOH approved laboratory, if possible. Interpretive Data: Reference Interval: By report	0098378, 170HPROGU	
gonadotropin medications, if possible, for at least 48 hours prior to collection of specimen. Collect: 24-hour urine. No special preservatives required. Specimen Preparation: Mix specimen well. Refrigerate during collection. Transfer 10 mL aliquot of urine to ARUP standard transport tubes Standard Transport Tubes. (Min: 5 mL) Submit total volume. Test is not performed at ARUP; separate specimens must be submitted when multiple tests are ordered. Transport Temperature: Frozen. On dry ice is preferred. Separate specimens must be submitted when multiple tests are ordered. Unacceptable Conditions: Remarks: Stability: Ambient: 1 hour; Refrigerated: 4 days; Frozen: 6 months Methodology: Quantitative Radioimmunoassay (RIA) Performed: Varies Reported: 7-13-10 days Note: CPT Codes: 83498 New York DOH Approval Status: Specimens from New York clients will be sent out to a New York DOH approved laboratory, if possible. Interpretive Data:	Specimen Requirements:	
Specimen Preparation: Mix specimen well. Refrigerate during collection. Transfer 10 mL aliquot of urine to ARUP standard transport tubes Standard Transport Tubes. (Min: 5 mL) Submit total volume. Test is not performed at ARUP; separate specimens must be submitted when multiple tests are ordered. Transport Temperature: Frozen. On dry ice is preferred. Separate specimens must be submitted when multiple tests are ordered. Unacceptable Conditions: Remarks: Stability: Ambient: 1 hour; Refrigerated: 4 days; Frozen: 6 months Methodology: Quantitative Radioimmunoassay (RIA) Performed: Varies Reported: 7-1310 days Note: CPT Codes: 83498 New York DOH Approval Status: Specimens from New York clients will be sent out to a New York DOH approved laboratory, if possible. Interpretive Data: Reference Interval:	Patient Preparation:	gonadotropin medications, if possible, for at least 48 hours
mL aliquot of urine to ARUP standard transport tubes Standard Transport Tubes. (Min: 5 mL) Submit total volume. Test is not performed at ARUP; separate specimens must be submitted when multiple tests are ordered. Transport Temperature: Frozen. On dry ice is preferred. Separate specimens must be submitted when multiple tests are ordered. Unacceptable Conditions: Remarks: Stability: Ambient: 1 hour; Refrigerated: 4 days; Frozen: 6 months Methodology: Quantitative Radioimmunoassay (RIA) Performed: Varies Reported: 7-1310 days Note: CPT Codes: 83498 New York DOH Approval Status: Specimens from New York clients will be sent out to a New York DOH approved laboratory, if possible. Interpretive Data: Reference Interval:	Collect:	24-hour urine. No special preservatives required.
submitted when multiple tests are ordered. Unacceptable Conditions: Remarks: Stability: Ambient: 1 hour; Refrigerated: 4 days; Frozen: 6 months Methodology: Quantitative Radioimmunoassay (RIA) Performed: Varies Reported: 7-1310 days Note: CPT Codes: 83498 New York DOH Approval Status: Specimens from New York clients will be sent out to a New York DOH approved laboratory, if possible. Interpretive Data: Reference Interval:	Specimen Preparation:	mL aliquot of urine to ARUP <u>standard transport tubes</u> Standard <u>Transport Tubes</u> . (Min: 5 mL) Submit total volume. Test is not performed at ARUP; separate specimens must be submitted
Remarks: Stability: Ambient: 1 hour; Refrigerated: 4 days; Frozen: 6 months Methodology: Quantitative Radioimmunoassay (RIA) Performed: Varies Reported: 7-1310 days Note: CPT Codes: 83498 New York DOH Approval Status: Specimens from New York clients will be sent out to a New York DOH approved laboratory, if possible. Interpretive Data: Reference Interval:	Transport Temperature:	
Stability: Ambient: 1 hour; Refrigerated: 4 days; Frozen: 6 months Methodology: Quantitative Radioimmunoassay (RIA) Performed: Varies Reported: 7-13+0 days Note: CPT Codes: 83498 New York DOH Approval Status: Specimens from New York clients will be sent out to a New York DOH approved laboratory, if possible. Interpretive Data: Reference Interval:	Unacceptable Conditions:	
Methodology: Quantitative Radioimmunoassay (RIA) Performed: Varies Reported: 7-1310 days Note: CPT Codes: 83498 New York DOH Approval Status: Specimens from New York clients will be sent out to a New York DOH approved laboratory, if possible. Interpretive Data: Reference Interval:	Remarks:	
Performed: Varies Reported: 7-1310 days Note: CPT Codes: 83498 New York DOH Approval Status: Specimens from New York clients will be sent out to a New York DOH approved laboratory, if possible. Interpretive Data: Reference Interval:	Stability:	Ambient: 1 hour; Refrigerated: 4 days; Frozen: 6 months
Reported: 7-1310 days Note: CPT Codes: 83498 New York DOH Approval Status: Specimens from New York clients will be sent out to a New York DOH approved laboratory, if possible. Interpretive Data: Reference Interval:	Methodology:	Quantitative Radioimmunoassay (RIA)
Note: CPT Codes: 83498 New York DOH Approval Status: Specimens from New York clients will be sent out to a New York DOH approved laboratory, if possible. Interpretive Data: Reference Interval:	Performed:	Varies
CPT Codes: 83498 New York DOH Approval Status: Specimens from New York clients will be sent out to a New York DOH approved laboratory, if possible. Interpretive Data: Reference Interval:	Reported:	7- <u>13</u> +0 days
New York DOH Approval Status: Specimens from New York clients will be sent out to a New York DOH approved laboratory, if possible. Interpretive Data: Reference Interval:	Note:	
York DOH approved laboratory, if possible. Interpretive Data: Reference Interval:	CPT Codes:	83498
Reference Interval:	New York DOH Approval Status:	•
	Interpretive Data:	
By report	Reference Interval:	
	By report	

Effective Date: August 5, 2024