

0098378

17-Hydroxyprogesterone, Urine

17OHPROGU

Performed: Varies**Reported:** 7-10 days

Specimen Required: Patient Prep: Patient should not be on any corticosteroid, ACTH, estrogen, or gonadotropin medications, if possible, for at least 48 hours prior to collection of specimen.

Collect: 24-hour urine. No special preservatives required.

Specimen Preparation: Mix specimen well. Refrigerate during collection. Transfer 10 mL aliquot of urine to ARUP Standard Transport Tubes. (Min: 5 mL) Submit total volume.

Storage/Transport Temperature: Frozen. On dry ice is preferred. Separate specimens must be submitted when multiple tests are ordered.

Stability (collection to initiation of testing): Ambient: 1 hour; Refrigerated: 4 days; Frozen: 6 months