

TEST CHANGE

Protein S Free, Antigen

0098894, PRO S FREE

Specimen Requirements:

Patient Preparation:

Collect: Light blue (sodium citrate). ~~Special~~Refer to Specimen Collection and Handling Hemostasis/Thrombosis Specimens guide located at <https://www.aruplab.com/Specimen-Handling/SpecialSpecimenCollection/Hemostasis-Thrombosis.pdf> for hemostasis/thrombosis specimen handling guidelines.

Specimen Preparation: Transfer 1.5 mL platelet-poor plasma to an ARUP standard transport tube. (Min: 1 mL)

Transport Temperature: CRITICAL FROZEN. Separate specimens must be submitted when multiple tests are ordered.

Unacceptable Conditions: Serum. EDTA plasma, clotted or hemolyzed specimens.

Remarks:

Stability: Ambient: 4 hours; Refrigerated: Unacceptable; Frozen ~~at -20 Degrees C: 3 months, at -70 Degrees C: 6 months~~

Methodology: Microlatex Particle-Mediated Immunoassay

Performed: Mon-Sat

Reported: 1-3 days

Note:

CPT Codes: 85306

New York DOH Approval Status: This test is New York DOH approved.

Interpretive Data:

Refer to report

Reference Interval:

Age	Male	Female
1-89 days	15-55%	15-55%
90-179 days	35-92%	35-92%
180-364 days	45-115%	45-115%
1-5 years	62-120%	62-120%
6-9 years	62-130%	62-130%
10-17 years	60-140%	60-140%
18 years and older	74-147%	55-123%